

CASE CONTEST: WOUND

Marshall Animal Care Center

Species and Breed	8 year old, F/S Dachshund
Presentation	Attacked by large dog; presented to second veterinarian 6 days post-attack after large amount of skin sloughed
Approximate Size of Area Treated (in cm ²)	387cm ²
Total Energy Delivered (= Power x Time)	Treated with 1740 J total (4.5J/cm ²) each session
Frequency of Treatment	Treated every-other-day. Dilute Nolvasan used to soak rolled gauze covered with Vet Flex EZ Tear cohesive bandage between laser sessions. No further surgical debridement and/or skin grafting was needed/performed

Photos and case courtesy of Marshall Animal Care Center, Marshall, MI



< Day 1 - 3/23 >



Day 44 5/7



18 total treatment